


accessibility EQUIPMENT



accessibility **equipment**

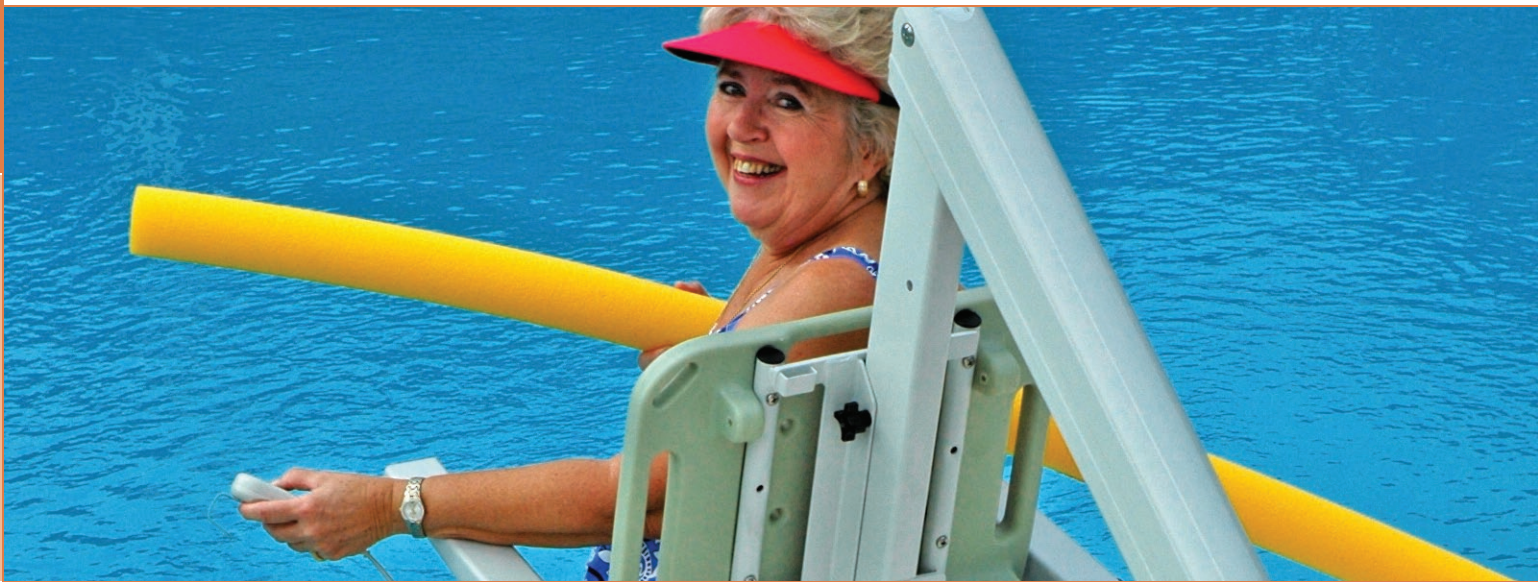

MORE ACCESSIBLE POOLS

Access to aquatic environments is vitally important to the quality of life for both mobility challenged individuals and seniors. A pool provides the optimal setting for low impact exercise or therapy. Fear of falling and other traditional land-based concerns virtually disappear when exercise is moved into the pool. As our society ages the mobility challenged population increases, as does the need for safe and accessible pools. Our line of powered lifts can help make pools more accessible starting today.



BATTERY & CHARGER

All pool lifts manufactured by S.R.Smith are powered by a 24 volt, rechargeable battery. This is a lead acid battery that should be left on the charger when not in use. It is IMPORTANT that the battery not be allowed to fully discharge, as this could damage the battery. The best practice is to make it a rule to put the battery on the charger each evening. If the lift is being stored for the winter, leave the battery on the charger all winter. You cannot overcharge the battery. The charger connects to the port located on the same side of the battery as the control box connection. Works with PAL, Splash! and aXs lifts.

residential



aXs Lift **NEW**

The aXs is a perfect low profile lift that blends in with today's backyard décor.

- ADA-compliant
- 300 lb/136 kg lifting capacity
- Semi-permanent installation—easily removable from anchor
- Powered by a 24-volt rechargeable battery
- User operable via waterproof hand control
- 360° rotation
- Made from powder coated stainless steel and aluminum
- Includes battery, charger, battery console cover, waterproof control, stainless steel anchor socket with cover and spanner key, footrest and seat belt assembly

commercial



PAL Series

Since its introduction in 1997, the PAL Portable Aquatic Lift has become the industry standard for providing access to swimming pools. The completely portable PAL is the perfect accessory for any commercial aquatic facility.

- ADA-compliant
- 300 lb/136 kg lifting capacity
- Completely portable—no deck fixtures and easy to move by one person
- Conveniently stores away when not in use
- Powered by a 24-volt rechargeable battery
- User operable via waterproof hand control
- 240° rotation
- Made from powder coated stainless steel and aluminum
- Comes with battery, charger, battery console cover, waterproof control, footrest and seatbelt assembly

Also available:

- **PAL Hi/Lo**—A variation of the PAL useful for facilities that have both in-ground pools and above ground spas.
- **PAL Spa**—Designed for above ground pools and spas up to 48 in/122 cm above the pool deck.



Splash! Series

This semi-portable lifting system is designed for public facilities seeking to provide user-friendly access to their swimming pools.

- ADA-compliant
- 400 lb/181 kg lifting capacity*
- Semi-permanent installation—easily removable from anchor
- Powered by a 24-volt rechargeable battery
- User operable via waterproof hand control
- 359° rotation
- Made from powder coated stainless steel and aluminum
- Includes battery, charger, battery console cover, waterproof control, stainless steel anchor socket with cover & spanner key, footrest and seat belt assembly

*Lifting capacity of the Extended Reach model is 300 lb/136 kg

Also available:


- **Splash! Hi/Lo**—A variation of the Splash! useful for facilities that have both in-ground pools and above ground spas.
- **Splash! Spa**—Designed for use with spas up to 48 in/122 cm above the deck.
- **Splash! Extended Reach**—For use with pools with unusual gutter configurations requiring a longer reach to the water line.
- **Splash! Mariner**—Designed to transfer the user from a dock to a boat, it uses a sling instead of a rigid seat, to eliminate physical transfers.

accessories




Stability Vest

The Stability Vest was designed to facilitate the transfer of individuals with limited or no independent upper body or trunk stability. The vest attaches the user to the chair, strapping them in for support.

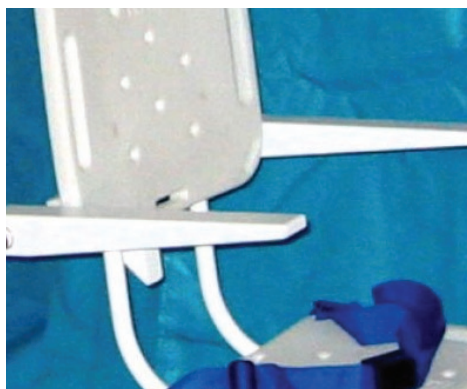

Arm Rests

Optional, flip-up armrests are available for all lifts. This accessory item can be retrofitted to a lift after it has been delivered. The armrests are available in white, for PAL and Splash models, and in gray, for the aXs.

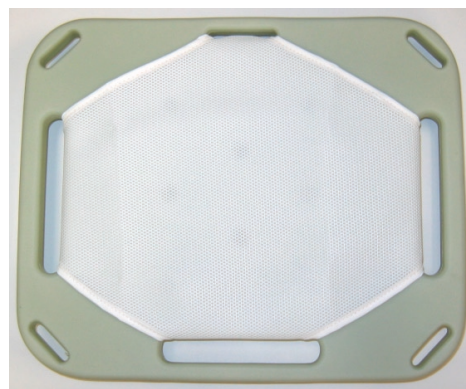

Seat Pad

White, waterproof seat pad designed to enhance comfort and minimize potential skin damage that could occur during transfer is available as an optional accessory.



Spine Board Attachment

The Spine Board Attachment works with any standard spine board (not included). Simply remove the chair assembly by pulling the attachment pin and replace that assembly with the Spine Board Attachment. One person can easily extract an injured swimmer from a pool, keeping the injured person fully supine during the procedure.




Splash! Caddy

The Splash! Caddy facilitates handling, transporting and storing a Splash! aquatic lift. Fabricated from rugged steel for strength and powder-coated for appearance, the Splash! Caddy helps to protect both your investment and your staff. The Caddy simplifies both insertion and removal of the connector shaft into the insert in the pool deck and becomes a convenient storage rack for the lift when not in use.



Battery Console Covers for PAL and Splash! Models

Made of Solution Dyed Polyester

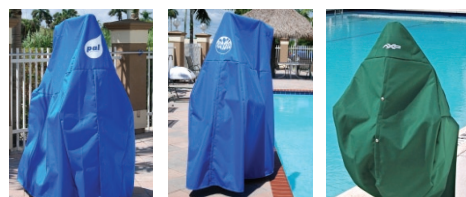

Custom Covers for PAL, Splash!, and aXs Models

Protect your investment with a full, all-weather cover, made from Solution Dyed Polyester. Available for PAL, Splash!, and aXs models.