



Range of pool lift with body support versus distance to pool edge

The attached table provides a clear, simple overview of the situations where the pool lift can be used in combination with a body support, based on the following factors:

- the distance between the socket and the edge of the pool (A)
- the water level relative to the top edge of the pool (B)
- the position of the user relative to the edge of the pool (C): standard value (minimum) = 40 cm

A (in cm)	C (in cm)					
	B = 0 cm	B = 10 cm	B = 20 cm	B = 30 cm	B = 40 cm	B = 50 cm
15	103	99	96,5	88	81	69
20	98	94	91,5	83	76	64
25	93	89	86,5	78	71	59
30	87	84	81,5	73	66	54
35	82	79	76,5	68	61	49
40	77	74	71,5	63	56	44
45	72	69	66,5	58	51	x
50	67	64	61,5	53	46	x
55	62	59	56,5	48	41	x
60	57	54	51,5	43	x	x
65	52	49	46,5	x	x	x
70	47	44	41,5	x	x	x
75	42	x	x	x	x	x

