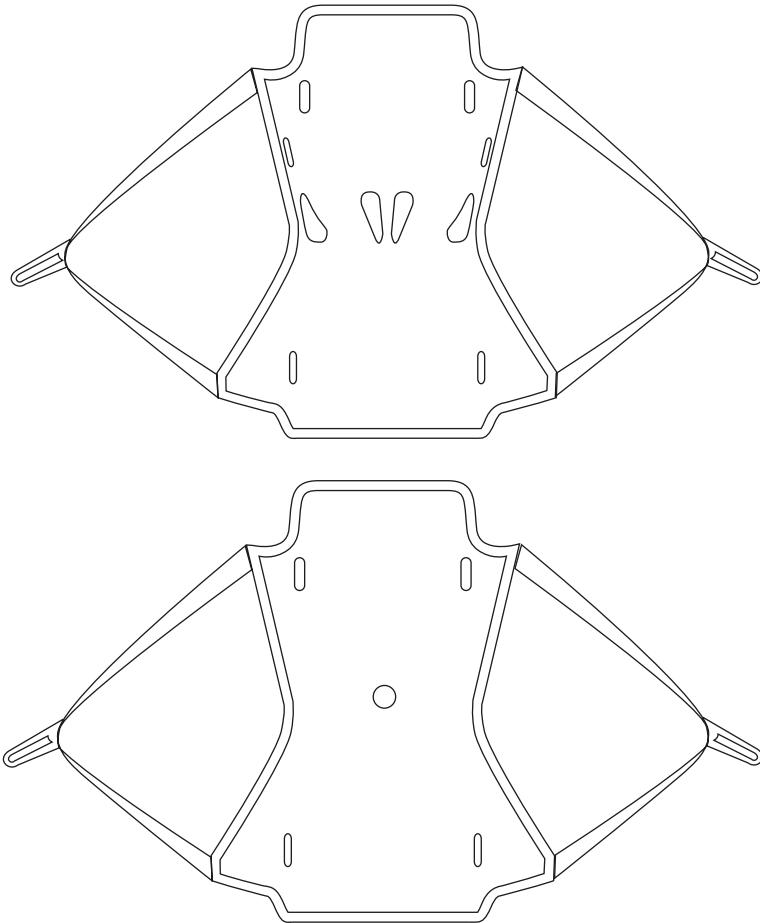
SEAT SLING (BATH/POOL)**TILSTOEL (BAD/ZWEMBAD)****HANDI-SIÈGE (BAIN/PISCINE)****BADESITZ (BAD/SCHWIMMBAD)****L'IMBRACATURA A SEDIA (BAGNO/ PISCINA)****SILLA DE BAÑO/PISCINA**

User guide - Handleiding - Mode d'emploi - Bedienungsanleitung Istruzioni per l'uso - Manual de usuario

Read Thoroughly Before Operating. Have your system inspected at least yearly. See inside user guide for details. Images may differ from reality.

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Slings	Spreaderbars							
	1 point	2 point				4 point		
2112 carabine hook	2117	Mobile hoists 1630/1641	2116	2115	2110	2120/2125	2110+2111	2140/2145
3404z.xxxx								
3489z.xxxx	✓	✓	✓	✓	✓	✓	✓	
3461z.xxxx	✓	✓	✓	✓	✓	✓	✓	
3415z.xxxx	✓	✓	✓	✓	✓	✓	✓	
3464z.xxxx	✓	✓	✓	✓	✓	✓	✓	
3418z.xxxx	✓	✓	✓	✓	✓	✓	✓	
3491z.xxxx	✓	✓	✓	✓	✓	✓	✓	
3481z.xxxx/3484z.xxxx/3487z.xxxx	✓	✓	✓	✓	✓	✓	✓	✓
3424z.xxxx/3431z.xxxx/3437z.xxxx	✓	✓	✓	✓	✓	✓	✓	✓
3451z.xxxx	✓	✓	✓	✓	✓	✓	✓	✓
3499z.xxxx	✓	✓	✓	✓	✓	✓	✓	✓
3445z.xxxx	✓	✓	✓	✓	✓	✓	✓	✓
3494z.xxxx	✓	✓	✓	✓	✓	✓	✓	✓
3540z.xxxx to 3545z.xxxx	✓	✓	✓	✓	✓	✓	✓	✓
3473.zxxx/3478.zxxx (1size)	✓	✓	✓	✓	✓	✓	✓	
3471z.xxxx	✓	✓	✓	✓	✓	✓		
3479z.xxxx	✓	✓	✓	✓	✓	✓	✓	
3497.zxxx (1size)	✓	✓	✓	✓	✓	✓	✓	✓
3498.zxxx (1size)						✓	✓	
3512z.xxxx								✓
3517z.xxxx								✓

z = size S, M, L / x = 0000 or 9000

Slings	Spreaderbars							
	1 point	2 point				4 point		
		2112 carabine hook	2117	Mobile hoists 1630/1641	2116	2115	2110	2120/2125
3461z.xxxx	✓	✓	✓	✓	✓	✓	✓	✓
3464z.xxxx	✓	✓	✓	✓	✓	✓	✓	✓
3481z.xxxx/ 3484z.xxxx/ 3487z.xxxx	✓	✓	✓	✓	✓	✓	✓	✓
3424z.xxxx/ 3431z.xxxx/ 3437z.xxxx	✓	✓	✓	✓	✓	✓	✓	✓
3445z.xxxx	✓	✓	✓	✓	✓	✓	✓	✓
3512z.xxxx								✓
3517z.xxxx								✓

z = size XS, XL / x = 0000 or 9000

Slings	Spreaderbars							
	1 point	2 point				4 point		
		2112 carabine hook	2117	Mobile hoists 1630/1641	2116	2115	2110	2120/2125
3530z.xxxx	✓	✓	✓	✓	✓	✓	✓	✓

z = size XXL / x = 0000 or 9000

The above mentioned slings can be ordered with client specific changes according to the allowed specifications mentioned in the Handi-Move manual for ordering such slings.

These slings are always based on the pattern of the slings mentioned in the above tables.

In these cases the slings will be delivered with the same reference number as the related sling adding suffix 9000

(For example : 3461S toilet sling with client specific changes will be delivered as 3461S 9000)

In conformity with

Konform richtlijn
Conforme la directive
Gemäss Richtlinie

93/42/EEG as amended by 2007/47/EG (Class I Medical Devices)
2011/65/EU (RoHS directive)

For Handi-Move International

Stefan Van Raemdonck bvba
Director Handi-Move international
05/01/2015

Content

1.	Safety instructions	2
2.	Field of application.....	3
3.	Product information.....	3
3.1.	Sling	3
4.	Explanation of the symbols	4
4.1.	Symbols on the label	4
5.	User instructions	5
5.1.	Fitting sling	5
5.2.	Combination with Handi-Move SureHands® Spreader Bars	7
6.	Maintenance.....	7
6.1.	Cleaning the sling.....	7
7.	Inspection.....	7
7.1.	Inspection by the user (prior to each use).....	7
7.2.	Annual inspection.....	7
8.	Technical specifications.....	8
9.	Warranty	9

1. Safety instructions

- Read and understand this user guide and the relevant instructions in the user guide for your Handi-Move lifting system before using.
- Before every use, ensure that the sling is suitable for the person concerned.
- Before you start lifting, make sure the person is positioned safely and the loops are attached properly.
- Keep the person's arms inside the sling.
- Never lift the person higher than necessary.
- When assisting, never leave the person alone in the sling.
- For sling maintenance, observe the washing instructions on the sling label.
- Before each use, check the fabric, seams, stitching, loops, loop attachments, for possible wear, damage or fraying. If you find wear or damage, replace the sling immediately and discard the old sling.
- The seat sling is suitable for people with sufficient sitting balance. The person should be able to move between his or her (wheel)chair and chair independently or with an aid or passively (e.g. By means of lifting equipment).
- If, for whatever reason, there may be a possibility that the user slides out of the sling, you should use the optional safety belt(s). If use of the safety belt(s) does not guarantee complete security, do not use the seat sling. The optional safety belt (#3473 1000) does not come standard with the seat sling.
- The approved combinations of SureHands® Spreader Bars/slings are the result of extensive testing in the interest of user safety. Using other slings, spreader bars and lift systems in combination with Handi-Move slings or SureHands® Spreader Bars (and vice versa) is potentially dangerous, may cause injury and is therefore not allowed.

NOTE!

THESE REGULATIONS FALL WITHIN THE SCOPE OF THE EUROPEAN DIRECTIVE ISO 10535 (HOISTS FOR THE TRANSFER OF DISABLED PERSONS - REQUIREMENTS AND TEST METHODS).

NOT FOLLOWING THE AFOREMENTIONED INSTRUCTIONS MAY LEAD TO PERSONAL OR THIRD PARTY INJURIES.

**THE MAXIMUM PERMISSIBLE LOAD MAY VARY BETWEEN THE CEILING MOTOR, SPREADER BAR AND SLING.
THE LOWEST PERMISSIBLE LOAD MUST THEN BE REGARDED AS THE MAXIMUM PERMISSIBLE LOAD.**

2. Field of application


The sling is used in hospitals, rehabilitation centers, geriatric centers, care homes, day centers and private homes. The sling can be used for transfers to and from: bed, whweldadeelchair, (reclining) chair, sofa, bath, pool, floor, treatment table...

3. Product information

3.1. Sling

This user guide refers to the following parts on your sling:

- A** Back section
- B** Seat section
- C** Loops
- D** Yellow strap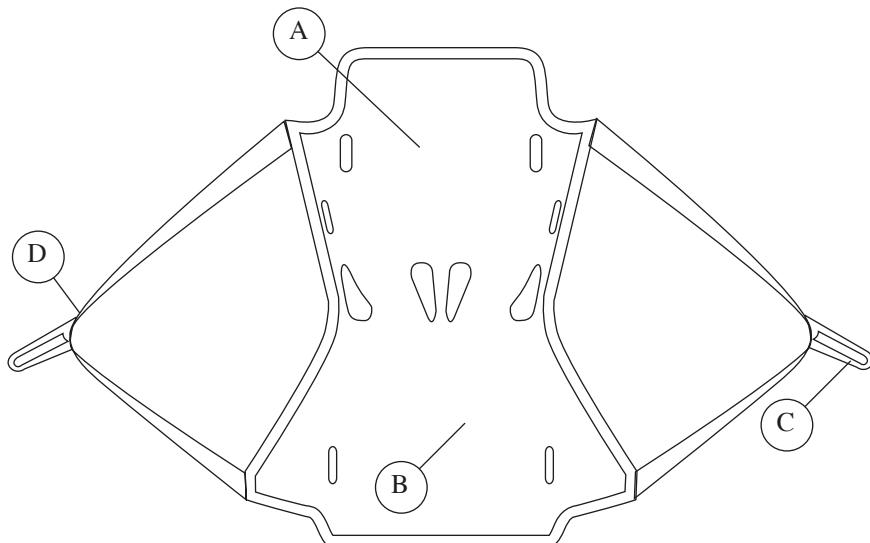

Remark

The model shown in the drawing can differ from the actual model.

This manual covers both seat slings for bath and pool


4. Explanation of the symbols

4.1. Symbols on the label

	Please read the user guide carefully before using the device.
	The device complies with the Medical Device Directive 93/42/EEC, as amended by Directive 2007/47/EC.
	Handwash at a water temperature between 30°C/86°F and maximum 40°C/104°F.
	Do not bleach, only use bleach-free washing products
	No dry cleaning. No professional dry cleaning. Do not use solvent-based stain remover.
	Do not tumble dry
	Do not iron. Irreversible changes may occur when using an iron.
	Combination spreader bar - sling allowed
	Combination spreader bar - sling not allowed

Remark

The labels shown can differ from the actual labels on your sling.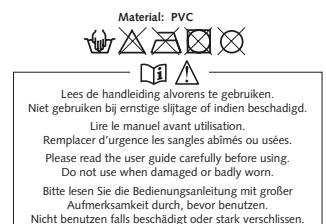

manufacturer:
Handi-Move International
Ten Beukerboom 13, 5400 Nijmegen
Belgium, Europe
info@handimove.com
www.handimove.com

Article No. 3473


Spreaderbar	OK / not OK
	✓
	✓
	✓
	✗
	✓
	✓
	✓
	✓
	✗

5. User instructions


5.1. Fitting sling

Use from a seated position


- step 1. Place the seat sling on the chair. The longest side should be positioned against the back of the chair, the shortest on the seat itself. The label is at the rear of the back section.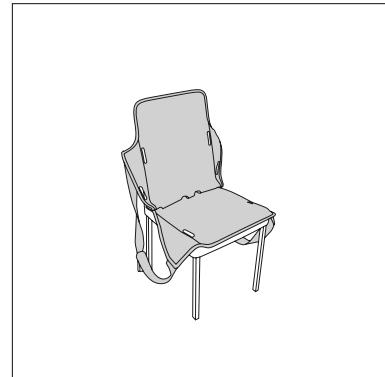

- step 2. Move the person from (wheel)chair to chair.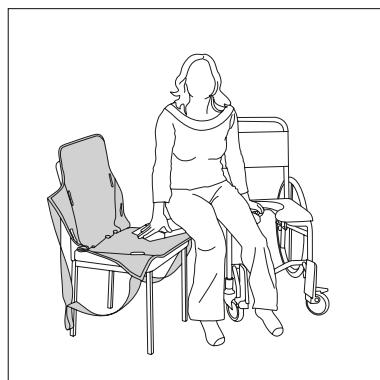

- step 3. Attach the loops to the spreader bar and lift.

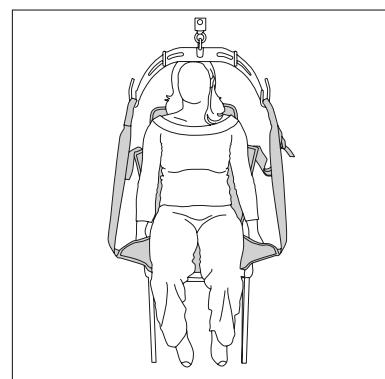


USER GUIDE

SEAT SLING (BATH/POOL)

step 4. The lifting position is determined by how much you tighten the yellow strap.



- Fully tightened = sitting position.
- Not tightened = half-lying position.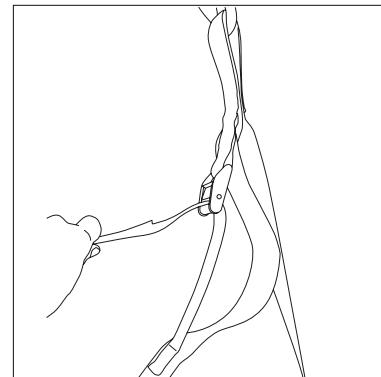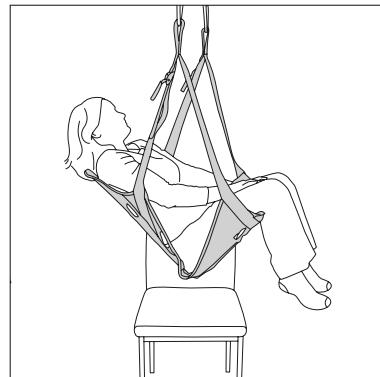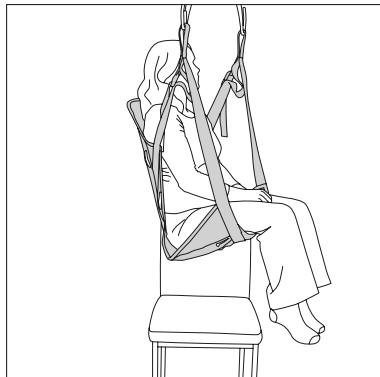

step 5. Seat sling in combination with safety belt around abdomen and/or chest.

Pull the two ends of the safety belt^(*) through the two openings on the side of the seat sling from the back and fasten them on the front. Tighten.

* Option: safety belt (#3473 1000)

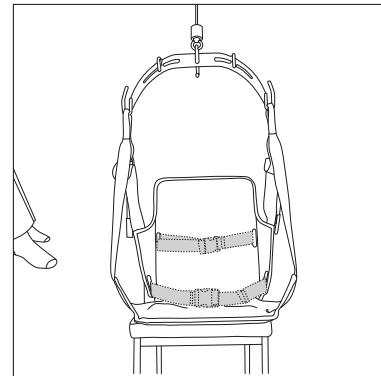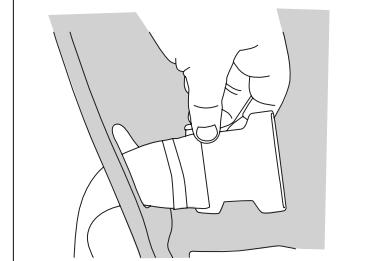NOTE!

ON THE SEAT SLING FOR BATH (#3478) THE BACK AND SEAT SECTION ARE WELDED. THEREFORE THIS SEAT SLING CAN'T BE USED IN A POOL BECAUSE IT CANNOT BE SUBMERGED. THE SEAT SLING FOR BATH COMES WITH A RED EDGE FOR EASY RECOGNITION. FOR USE IN A POOL YOU NEED TO USE THE SEAT SLING FOR POOLS (#3473).

5.2. Combination with Handi-Move SureHands® Spreader Bars

	Single point suspension	Double point suspension				Four point suspension			
	2112 Carabiner	2117 small	SureHands® mobile lift systems 1630 / 1641	2116 medium	2115 large	2110	2120/2125	2110+2111	2140/2145
Seat sling (bath/pool)	*	*	***	***	***	***	**	∅	∅

*** preferred use

** generally good use

* use not advisable due to reduced comfort

∅ use not permitted due to safety risks

6. Maintenance

6.1. Cleaning the sling

Follow the instructions on the label of the sling

Handwash at a water temperature between 30°C/86°F and maximum 40°C/104°F.

Do not bleach, only use bleach-free washing products

No dry cleaning. No professional dry cleaning. Do not use solvent-based stain remover.

Do not tumble dry

Do not iron. Irreversible changes may occur when using an iron.

7. Inspection

7.1. Inspection by the user (prior to each use)

- Before each use, check the fabric, seams, stitching, loops, loop attachments, for possible wear or damage. If you find wear or damage, replace the sling immediately and discard the old sling.

7.2. Annual inspection

- The sling must be inspected at least once a year.
- When using the sling intensively and washing it often, it is advisable to inspect the sling more frequently.
- Inspection may only be carried out by Handi-Move or a technician approved by Handi-Move.
- After the inspection, a report will be prepared documenting the tests that were carried out and the parts that need to be replaced.

USER GUIDE

SEAT SLING (BATH/POOL)

Contact your Handi-Move distributor for annual maintenance and advice.

Address of a Handi-Move distributor in your area:

Handi-Move nv, Ten Beukenboom 13, 9400 Ninove, Belgium

Tel. + 32 54 31 97 10

Fax + 32 54 32 58 27

E-mail: info@handimove.com

Website: www.handimove.com

8. Technical specifications

Grey material 100% pvc

Lifting loops 100% polyester

Expected lifetime 1 to 5 years

Lifetime of this sling will depend largely on how intensively it is used, maintenance, etc...

Safe Working Load 225 kg / 500 lbs

9. Warranty

This equipment is sold under an exclusive warranty of two (2) years from date of purchase.

The manufacturer warranties the equipment to be free of defects in material and workmanship. We agree to repair or replace any products, or parts thereof, which have been returned to our factory within the stated warranty period, and which after examination shall disclose that the product is defective. Batteries are excluded. Transportation charges to our factory must be prepaid. Return charges for repaired or replaced products will be borne by the factory. Onsite labor charges are not covered by the manufacturer warranty. Only labor performed on equipment at the Manufacturer or Distributor's facilities will be covered by the manufacturer warranty.

This warranty does not apply if the equipment has been subject to misuse, neglect, accidents, incorrect wiring (not our own), improper installation, or put to use in violation of instructions furnished by us or repaired or altered by other than one of our factory authorized representatives without our prior written consent.

We reserve the right to modify or change the equipment in whole or in part, at any time prior to delivery in order to include refinements deemed appropriate by us, but without incurring any liability to modify or change any equipment previously delivered, or to supply new equipment in accordance with earlier specifications.

Loss or damage must be reported within 14 days after receipt of equipment. Visible loss of or damage to cartons must be noted at time of delivery on bill of lading or delivery receipt.

This warranty is expressly in lieu of all other warranties expressed or implied. We neither assume, nor authorize any other person to assume for us, any other liability in connection with the sale or use of this equipment. In no event shall we be liable for consequential or special damages. We make no warranty whatsoever in respect to accessories or parts not supplied by us. This warranty is valid to the original purchaser only. This warranty is not transferrable.

Note for the USA:

SUREHANDS has offered special warranty terms to certain Government and State Agencies and Funding Sources. If your system is funded by one of these entities, your system will be covered by the terms of that warranty instead of the standard warranty written above.

For products purchased from either a Handi-Move Dealer or Distributor please refer to their warranty terms and conditions.

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