

Easytrack with Maxi Sky 440

ARJOHUNTLEIGH
GETINGE GROUP



The portable gantry that is designed to be assembled in minutes by just one person

The unique floor-to-ceiling design eliminates the need for specialist alterations or electrical installations. This allows the system to be used permanently, semi-permanently or even temporarily. The Easytrack is the ideal solution when a fixed ceiling track cannot be installed.

Assemble in Five Easy Steps

1 To lock in place against the ceiling, extend posts and rotate until red section is completely hidden.



2 Fix one end of rail onto the post.



3 Extend rail to required length and fix other end onto second post.



4 Fix end cap to lock in place.




5 Use of the reaching pole when attaching the Maxi Sky 440 removes the need to stand on steps or chairs. This powerful lightweight cassette has a safe working load of 200kg (31.5 stone) and is compatible with the ArjoHuntleigh range of loop slings and Flite patient specific loop slings.




The lightweight aluminium Easytrack comes in a portable kit form that can be easily transported along with the Maxi Sky 440 portable hoist. A useful carry bag is available for the Easytrack.

Also available is the Maxi Sky 440 Cart that stores the cassette, reaching pole and slings. The cassette is attached to the Easytrack directly from the cart, removing all manual lifting.

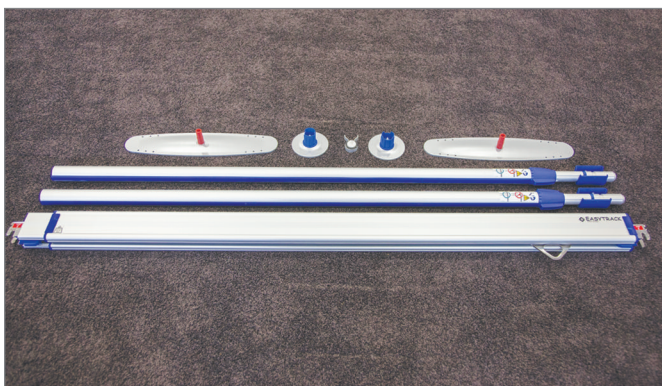

Features

- A multitude of unique safety systems have been incorporated into the Easytrack to ensure peace of mind. These systems include preventing the post losing grip if the floor or ceiling flexes, preventing the post being lowered if the rail is still connected or the rail disconnecting from the post.
- The small feet of the Easytrack allow use even in the smallest bedroom or bathroom where a conventional floor lifter could not be manoeuvred.
- Unlike regular gantry tracks the Easytrack can be extended to suit different locations as and when required. The track can be extended when in position to allow adjustment and then reduced to enable easier storage or transportation. The telescopic posts allow the Easytrack to be used with a variety of ceiling heights and give maximum transfer clearance height over beds and chairs.

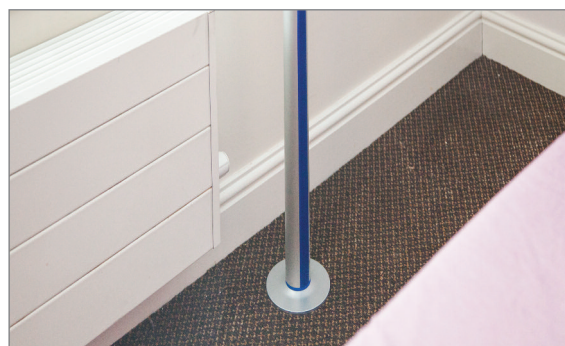

Our thanks to Lifestyle Care and particularly the team at Moorland Gardens in Luton for allowing us to photograph the Easytrack in situ in their Home.

ARJOHUNTLEIGH
GETINGE GROUP

ArjoHuntleigh UK – Head Office

ArjoHuntleigh House, Houghton Hall Business Park, Houghton Regis, LU5 5XF

Sales Phone: +44 (0) 1582 745700

Sales Fax: +44 (0) 1582 745745

Rental 24hr Helpline

Lo-call Phone: +44 (0) 8457 342000

Rental Fax: +44 (0) 1582 745843

Email: sales.admin@ArjoHuntleigh.com

www.ArjoHuntleigh.co.uk

ArjoHuntleigh UK – Service and Technical Centre

St Catherine Street, Gloucester, GL1 2SL

Phone: +44 (0) 8456 114114

Service Fax: +44 (0) 1452 525207

Email: ukservice@ArjoHuntleigh.com

www.ArjoHuntleigh.co.uk

ArjoHuntleigh UK – Belfast Office

19 Heron Road, Sydenham Business Park, Belfast, BT3 9LE, Northern Ireland

Phone: +44 (0) 28 9050 2000

Fax: +44 (0) 28 9050 2001

Email: belfast@ArjoHuntleigh.com

www.ArjoHuntleigh.com

ArjoHuntleigh (Ireland) Ltd.

EA House, Damastown Industrial Park, Mulhuddart, Dublin 15, Ireland

Phone: +353 (0)1 809 8960

Fax: +353 (0) 1 809 8971

Email: dublin@ArjoHuntleigh.com

www.ArjoHuntleigh.com

GETINGE GROUP

Getinge is a leading global medical technology company with operations in the areas of surgery, intensive care, infection control, care ergonomics and wound care.