

ULTIMATE STAIRLIFT



ULTIMATE JOURNEY



STEP 1

Lower the arms and the seat pad, which is linked to the footplate. Ensure the safety belt is fastened.

STEP 2

Press the rocker or push the joystick depending on your chosen control in the direction you want to go.



STEP 3

Sit back and enjoy the Ultimate journey, taking your freedom to the next level!



STEP 4

At the top of the stairs, use the sidemounted swivel lever to turn the chair or wait for the powered swivel to turn itself, and safely step off.



Once finished, fold up the arms and the seat, which will fold up the footrest.



The Ultimate experience is exactly that. It has changed lives for the better






ULTIMATE FEATURES

The ergonomic seat by Platinum is one of the most advanced stairlift chairs available. Designed with the user in mind the seat comes with a wealth of built-in features.





120kg weight capacity with 160kg Heavy duty option

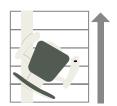

Upward facing travel



75° incline




Two way power swivel available



Choice of controls



Interchangable controls

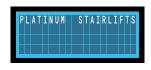

Digital display







ULTIMATE RAIL

The Ultimate is now an option for staircases previously unsuitable to fit stairlifts.

Clearing lower bulkheads and saving space on steep staircases, the Ultimate is the solution to many problems faced when fitting a stairlift.

A rail angle up to 75° and multiple bend radii (210mm to 500mm) allow the Ultimate rail to be designed to fit perfectly to the stairs.

> The diameter of the tube is 80mm. **Actual Size**





ULTIMATE RAIL

Starting options




Standard Start The rail starts on the landing before the stairs.



First Step Start Removes the need for the rail to intrude onto the landing.

Finishing options

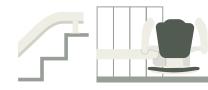

Standard Finish The rail can continue on the landing.



Flush Finish The rail can finish when the steps do.

Start and Finish options

Wrap Short, 90° or 180°

The wrap option can be used at both the top and bottom of the journey. Allowing the stairlift to wrap fully around the bannister, at a 90° turn to follow the wall or a tight turn.



ULTIMATE CHAIR

Choice is the important factor in this chair design. Adjustable arms and backrest allow the chair to fit you perfectly, maximising comfort. With a heavy duty option, the Ultimate will accommodate needs that other stairlifts cannot.

Enhancing the experience further; we have taken reassurance to the next level with a digital display for real time diagnostics on your stairlift.

UPHOLSTERY COLOURS











Upward Travel

Another industry first for The Ultimate is the upward facing travel. Allowing the user to face up the stairs while travelling. Not available with the first step start.





ULTIMATE MEASUREMENTS

